- DE Gebrauchsanweisung
GB Instructions for use
FR Mode d'emploi
ES Instrucciones de uso
IT Istruzioni per l'uso
PT Instruções de uso
NL Gebruiksaanwijzing
SE Bruksanvisning
DK Brugervejledning
NO Bruksanvisning
FI Käyttöohjeet
GR Οδηγίες Χρήσης
CZ Návod k použití
HU Használati utasítás
LV Lietošanas instrukcija
LT Naudojimo instrukcija
PL Instrukcja obsługi

Heraeus Kulzer GmbH
Grünweg 11
63450 Hanau (Germany)
info@heraeus.com
www.heraeus-dental.com
Importado e Distribuído por Heraeus Kulzer South América Ltda.
CNPJ 48.708.010/0001-02
R. Dr. Cândido Espinheira, 250 - São Paulo - SP - CEP 05004-000
sac@heraeus.com
Resp. Técnica: Dra. Regiane Marton - CRO 70.705
Nº ANVISA: vide embalagem

66040272/00 - D.O.G./01.09

Technische Informationen

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Technical Information

Table with 2 columns: Silicon based precision condensation curing impression material. and Instructions for use GB

Table with columns: Optosil®/Xantopren®, EN ISO 4823, Colours, Dosage*, Mixing-time, Working-time**, Time in mouth, Shore A hardness, Strain in compression, Recovery from deformation, Linear dimensional change

* At room temperature of 23°C/73°F, 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.

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Stand: 01.2009

User Advice
The Optosil®/Xantopren® assortment is a complete system. Components can be individually combined for all impression techniques.

Pouring the model:
Pouring can take place 30 minutes after the impression is removed from the mouth.

Recommended model material:
All commercially available modelling stones can be used.

Electroplating:
Disinfection:
Impressions should generally be treated by immersion for 10 minutes* in 5,25% Sodium Hypochlorite solution.

Adhesion:
Storage conditions:
Warning and precaution for Aktivator universal Plus:
Flammable. Inhalation can cause health problems.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved.

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